

Reusable Cannula

Attachment

2025.07.28



01 Manufacturer's name and address

- Nanum Company Co., Ltd.
- 5, Gomo-ro 41-gil, Suseong-gu, Daegu, Republic of Korea



02 Permission Information

- Manufacturing License No.: 4869
- Product License (Certification) No.: Jane 08-772
- Name (Product Name, Item Name, Model Name): General Purpose Cannula, NC1-19050L, and 2,867 other items
- QTY: 1 EA



03 Purpose of use

Inserted into a blood vessel or body cavity and connected to a connector or tube for the purpose of suction and injection




04 Storage or preservation method

- Store at room temperature



05 Contact information for reporting side effects

- National Institute of Medical Devices Safety Information
(080-080-4183)

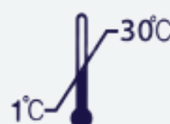

Reusable Cannula

Attachment

2025.07.28

06 How to use

A. Preparations before Use

- 1) Ensure that the catheter cannula in storage is clean and sterile enough for use.
- 2) Avoid contact with sterilized instruments by anyone not wearing sterile clothing.
- 3) Before use, inspect the instrument for any noticeable deformation, pinholes, or foreign matter.
- 4) Sterilization
 - ㉞ Sterilization is performed by autoclaving.
 - ㉟ Follow the sterilization method and use the equipment according to the manufacturer's instructions.
 - ㊱ Distribute the instruments in trays to minimize contact between them during sterilization and drying.

B. Instructions for Use

- 1) Use only for its intended purpose. Do not forcefully use the instrument for any other purpose.
- 2) If subjected to a severe impact or dropped, replace it with a different instrument.
- 3) If corrosion or cracks are discovered during use, immediately discontinue use and replace it with a different instrument.

D. Storage and Maintenance Methods after Use

- 1) Cleaning: After use, catheter cannulas should be soaked in water to prevent blood or foreign matter from drying and to remove any dried blood or foreign matter. Then, wash them by hand or machine.
- 2) Inspection
Check the cleaning status to ensure effective sterilization.

07 Precautions for use

- 1) Sterilization of instruments may cause staining, so follow the sterilizer manufacturer's instructions. Do not open the cover until the steam has completely dissipated.
- 2) If washing with water containing a high iron content, a rust-colored film may form. Consult a professional to assess the potential for water treatment issues before use.
- 3) Use only for its intended purpose and do not overuse for any other purpose.
- 4) If problems arise during storage or use, contact the manufacturer to take follow-up measures and prevent recurrence.

