

General-purpose catheter cannula
(MB-18100 and 2,080 other models)

INSTRUCTION FOR USE

Writing	Review	Approval

- For accurate and safe use, read the instruction manual carefully before use.
- It helps when there are any questions or checks during use.
- Be sure to read and use stability precautions accurately.
- Please make sure to keep it well after reading it.

00.

INDEX

01

Product Configuration and Key Names ----- 3

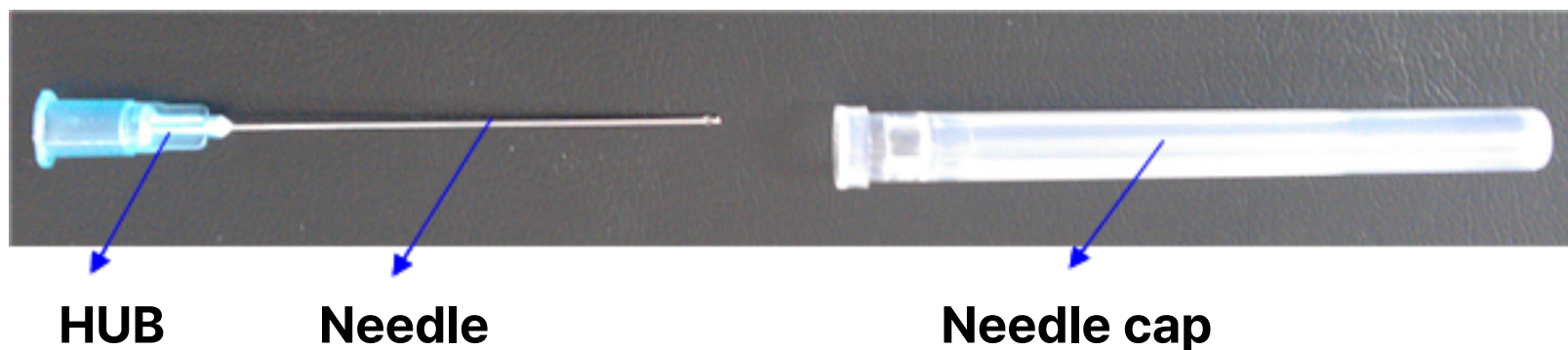
02

How to use and precautions ----- 4~5

03

Product Description ----- 6

Product Configuration and Key Names

**HUB**

Fix the needle and connect it to another instrument.

Needle

It is used to enter the body and inject drugs.

Needle cap

It is used as a needle cover and prevents damage to the needle.

How to use and precautions

- **Purpose of use**

It is connected to a connector or tube with a cannula that is inserted universally into a blood vessel or body cavity.

- **Pre-use Preparations**

1. This product is completely sterilized and sealed, so check whether the packaging is damaged or not.
2. If the packaging is damaged, do not use it.

- **How to use**

1. After checking the packaging status of the product, open it when there is no abnormality, and check whether there is any abnormality in the product.
2. Open the wrapper, take out the general-purpose catheter cannula, and use it in combination with other connecting devices.
3. Be sure to open it just before use.

- **How to store and manage after use**

1. This product is disposable, so it is never reused.
2. This product is disposable, so it is safely disposed of after use.

How to use and precautions

• Precautions for Use

1. Since this product is disposable, it should be discarded after use, and re-sterilization and reuse are prohibited.
2. Do not use products whose individual (single) packaging is opened or damaged because sterilization cannot be guaranteed.
3. If contaminated products are used, they can be transmitted to diseases such as various infectious diseases, so be careful.
4. Do not use cannulas that have been damaged or bent before use.
5. If foreign substances are found in the product before use, use is prohibited.
6. Prohibits use outside of general procedural purposes.

• Storage method and expiration date

1. Storage method: Store at room temperature and avoid direct sunlight and high temperature and humidity.
2. Expiration date: 3 years from the date of manufacture

• Related Inquiries on Side Effect Reporting

- Korea Medical Device Safety Information Service (080-080-4183)

Product Description

1. Product name: General-purpose catheter cannula(Disposable)
2. Model name (model name): MB-18100 and 2,080 other models
3. Trade name of manufacturer: Nanum Company Co., Ltd
4. Manufacturer's address: 5, Gomoro 41-gil, Suseong-gu, Daegu
5. Manufacturer's Contact: TEL)053-795-7076-7 / FAX)053-795-7078
6. Manufacturing business license number: No. 4869
7. Item license number: Jane 13-501
8. Packaging unit: Company packaging unit

※ This product is a medical device