

General Purpose Cannula(Disposable)

Attachment

2025.07.28



01 Manufacturer's name and address

- Nanum Company Co., Ltd.
- 5, Gomo-ro 41-gil, Suseong-gu, Daegu, Republic of Korea



02 Permission Information

- Manufacturing License Number: No. 4869
- Product License (Certification) Number: Jane 13-501
- Name (Product Name, Product Name, Model Name):
Mirror Cannula, etc., General Purpose Cannula, MC-14200 and 2080 others
- QTY: 1EA



03 Purpose of use

A cannula that is generally inserted into a blood vessel or body cavity and is connected to a connector or tube.



04 Storage or preservation method

- Store at room temperature and avoid direct sunlight.
- Avoid hot and humid places.



05 Contact information for reporting side effects

- National Institute of Medical Devices Safety Information
(080-080-4183)



General Purpose Cannula(Disposable)

Attachment

2025.07.28

06 How to use



A. Preparation before Use

- 1) This product is completely sterilized and sealed, so check the packaging for damage.
- 2) Do not use if the packaging is damaged.

B. Directions for Use and Operation

- 1) Check the product packaging. If there are no abnormalities, open the package and inspect the product for any abnormalities.
- 2) Open the packaging, remove the general purpose cannula, and connect it to another connecting device for use.
- 3) Open the package immediately before use.

C. Storage and Management

- 1) This product is disposable. Never reuse.
- 2) This product is disposable. Dispose of safely after use.

07 Precautions for use



A. This product is disposable and must be discarded after use. Re-sterilization and reuse are prohibited.

B. Sterilization cannot be guaranteed for products with opened or damaged individual packaging, so do not use them.

C. Being pricked by a contaminated product can lead to various infectious diseases.

D. Do not use needles that are damaged or bent before use.

E. Do not use if foreign substances are found in the product before use.

F. Do not use for purposes other than general treatment purposes.

Disposable sterile medical devices

Do not reuse

MD

STERILE EO

