

Nanum Syringe
(NS-50 and 2 other models)

INSTRUCTION FOR USE

Writing	Review	Approval

- For accurate and safe use, read the instruction manual carefully before use.
- It helps when there are any questions or checks during use.
- Be sure to read and use stability precautions accurately.
- Please make sure to keep it well after reading it.

00.

INDEX

01

Product Configuration and Key Names ----- 3


02

How to use and precautions ----- 4~5

03

Product Description ----- 6


Product Configuration and Key Names

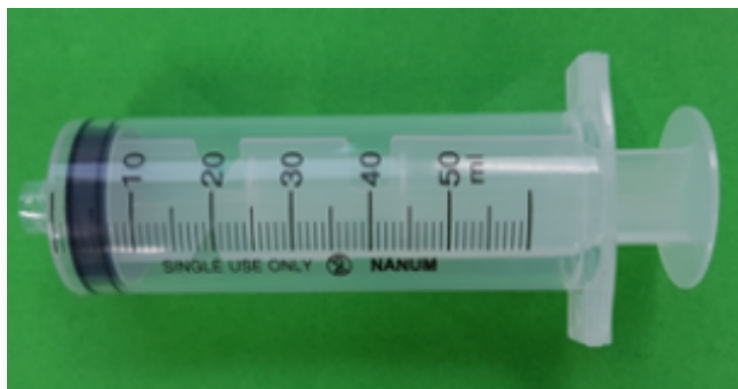

1) Luer Lock Clip TYPE



2) Luer Lock TYPE

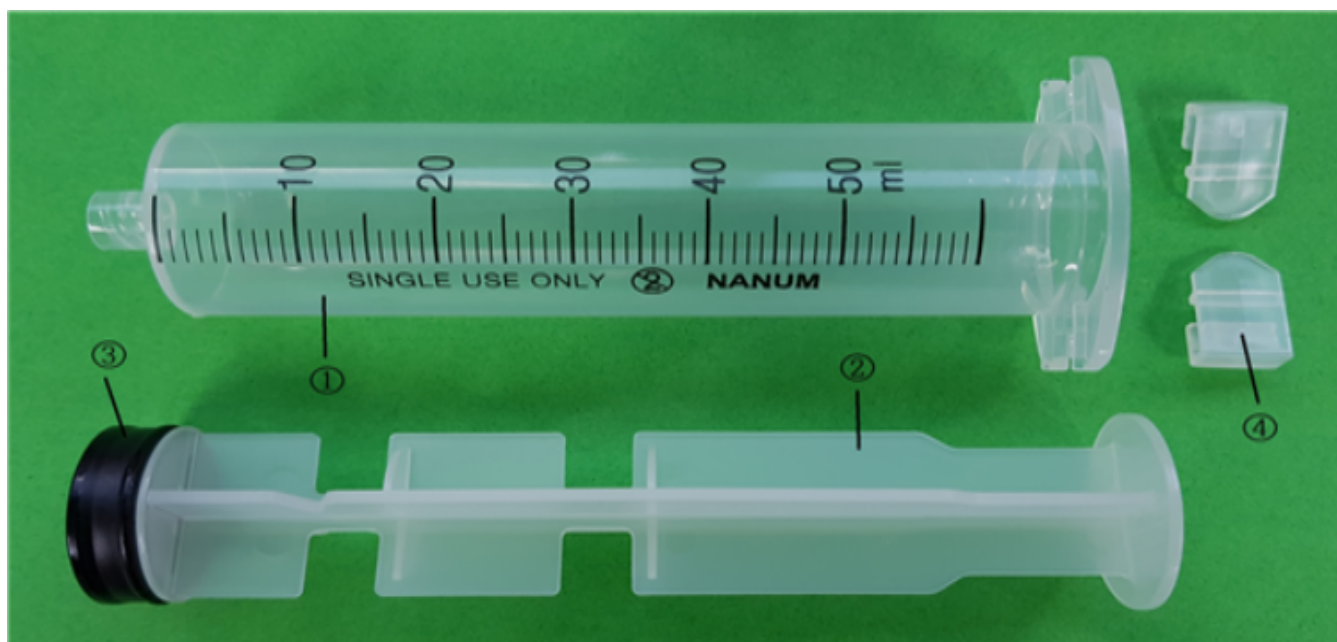

No	Part Name	Functional Description
①	Cylinder	It is a cylindrical container with a lock function at the end of the container coupled to the immersion, and a clip ④ is combined for a 50 ml outer handle.
②	Plunger	It serves to transmit the force necessary to advance and retreat the Gasket to the Gasket.
③	Gasket	It functions to inhale and release chemical solutions and prevents leakage of chemical solutions.
④	Clip	It is coupled to the outer cylinder handle and serves to fix the plunger.

How to use and precautions

• Purpose of use

Plastic sterile syringes used to inject medicines, etc. into the body and collect fat

• Pre-use Preparations

Before opening the product, check the presence or absence of abnormalities (missing parts, etc.) of the packaged syringe and the integrity of the packaging, and then open it when there is no abnormality.

• How to use

1. If you want to inject a drug

- Open the wrapper, take out the syringe, and select a general-purpose catheter cannula (separately certified product) of the appropriate gauge and connect it to the end of the barrel.
- Retreat the plunger to inhale the medicinal fluid, and then advance the plunger to remove the air in the syringe.
- After sterilizing the area cleanly, puncture the area with a separate certified product.
- Insert a syringe combined with a general-purpose catheter cannula (separately certified product) into the punctured area and press the plunger to inject it.
- The treatment area is compressed with disinfected gauze to stop the bleeding.

2. If you want to collect fat

- Insert a general-purpose catheter cannula combined with a syringe into the punctured area and retract the plunger to inhale fat and medication.
- In the case of a 50ml syringe with a clip attached, use the locking function (maintain negative pressure) to facilitate suction.
(For 50ml syringe, locking (maintaining negative pressure) is possible in 30ml and 50ml areas)
- When inhalation is complete, remove the cannula from the body and release the syringe lock to release the negative pressure. (It can be used as a syringe rear plug by breaking the groove in front of the push rod.)
- The treatment area is compressed with disinfected gauze to stop the bleeding.

How to use and precautions

- **How to store and manage after use**

1. This product is disposable, so it is never reused.
2. This product is disposable, so it is safely disposed of after use.

- **Precautions for Use**

1. Since this product is disposable, it should be discarded after use, and re-sterilization and reuse are prohibited.
2. This product is a medical device, and the purpose of use, the method of use, and the precautions should be fully understood and used.
3. Prohibits the use of expired products.
4. Do not use it when damage or foreign substances are found in the packaging before use.
5. Open just before use and use immediately after opening.
6. Prohibits use outside of general procedural purposes.

- **Storage method and expiration date**

1. Storage method: Store at room temperature and avoid direct sunlight and high temperature and humidity.
2. Expiration date: 3 years from the date of manufacture

- **Related Inquiries on Side Effect Reporting**

- Korea Medical Device Safety Information Service (080-080-4183)

Product Description

1. 제품명 : Nanum Syringe(Disposable)
2. Model name (model name): NS-50 and 2 other models
3. Trade name of manufacturer: Nanum Company Co., Ltd
4. Manufacturer's address: 5, Gomo-ro 41-gil, Suseong-gu, Daegu, Republic of Korea
5. Manufacturer's Contact: TEL)053-795-7076-7 / FAX)053-795-7078
6. Manufacturing business license number: No. 4869
7. Item license number: Jane 20-446
8. Packaging unit: Company packaging unit

※ This product is a medical device